

Accora 400 / Accora 600 Mattress

Instructions For Use



Introduction

DEALER: These instructions **MUST** be given to the user of the product.

USER: BEFORE using this product, read this manual and save for the future reference.

General

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING: Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION: Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

Product Description

The Accora 400 and 600 mattresses are multi-tiered pressure redistribution mattresses with high density head / torso sections. The mattress incorporates an air channel heel section, firm mattress edges and a 4 way stretch top cover.

Clinical Applications

Accora 400 is indicated for users up to 350 lbs. Accora 600 is indicated for users up to 450 lbs. Suitability for the mattress must be assessed by a competent practitioner.

Instructions for use

1. Remove all packaging.
2. Lay the mattress onto the bed frame, ensuring there are no sharp edges which may tear the cover.
3. Do not use the mattress without a cover as it provides a protective barrier which helps extend the life of the product.

4. Ensure the mattress is orientated with the silk screen text facing upwards and the feet graphic at the foot end of the bed.

Decontamination

WARNING

Mattresses MUST be cleaned regularly after each patient use. Because of the potential risk of infectious exposure, cleaning with the patient on the bed is not recommended. All equipment should be inspected. Any item that is visibly soiled with the patient's blood or other body fluids should be properly cleaned or removed. Staff members should treat all soiled bedding as if it were contaminated with pathogenic microorganisms. Staff members should wear appropriate protective clothing when cleaning mattresses. All cleaning solutions must be properly diluted according to the manufacturer's instructions. Follow standard institutional wipe down and infection control procedures. DO NOT open the zippered mattress top cover during cleaning, infection control procedures or during use. DO NOT launder or autoclave mattress covers.

To clean the mattress

1. Remove all bedding.
2. Clean the top and bottom covers with an organic cleaner or soapy water.
3. Wipe down the mattress unit with a damp cloth, pre-soaked with warm water containing a mild detergent. Approved intermediate level disinfectants may be used according to the cover material.
4. After cleaning, please avoid dust and proximity to dusty areas. All parts should be air dried thoroughly before use.

Accora 400 / Accora 600 Mattress

Instructions For Use



CAUTION

To avoid fabric breakdown and loss of waterproofing, use the following types of properly diluted standard cleaning solutions:

- Quaternary Wipe Down
- Phenolic Wipe Down
- 8-10% Bleach/Water Wipe Down

Iodophor type disinfectants (e.g. Betadine® etc.) are NOT recommended and will stain fabrics.

Maintenance

Inspection

The internal and external parts for the mattress must be inspected at least every 3 months for damage and wear. Remove the mattress from service and replace all damaged parts.

Inspection Record

An inspection record should be kept recording the Inspection date, Name of Inspector, Whether damage found, Date removed from service, repair date, returned to service date.

Compression Test


ENTRAPMENT WARNING

Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions, and mattress thickness, size or density could increase the risk of entrapment. Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment.

Refer to the owner's manuals for beds and rails for additional product and safety information.

Mattress MUST fit the bed frame and bed rails snugly to reduce the risk of entrapment.

1. Remove the bedding from the mattress.
2. Measure the thickness of the mattress (mattress depth) in an uncompressed area.
3. Lay a straight edge over the compressed area of the mattress. Measure the depth of the compressed area (compression depth).
4. Use the following formula to determine the amount of compression (compress percentage):

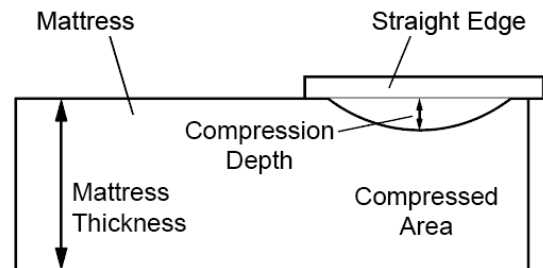

$$\frac{\text{Compression Depth}}{\text{Mattress Depth}} \times 100 = \text{Compress Percentage}$$

Disposal

Disposal of the mattresses should be in accordance with local regulations and facility standard operating procedures.

Limited Warranty

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state. Accora warrants the mattress and cover when purchased new and unused to be free from defects in materials, workmanship, and assembly for the following time periods:

Part No	Foam	Cover
ACC400-0-FM1-100	5 Years	2 Years
ACC400-0-FM1-200	5 Years	2 Years
ACC400-0-FM2-100	5 Years	2 Years
ACC600-0-FM1-100	10 Years	2 Years
ACC600-0-FM1-200	10 Years	2 Years
ACC600-0-FM2-100	10 Years	2 Years

If within such warranty period any such product shall be proven to be defective, such product shall be repaired or

Accora 400 / Accora 600 Mattress

Instructions For Use



replaced with new or refurbished product components, or are found corresponding to the proper service time shall be offered, at Accora option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Moreover, the warranty period of their paired or replaced product shall terminate with the termination of the warranty period for the original product component.

For warranty service, please contact the dealer from whom you purchased your Accora product. In the event you do not receive satisfactory warranty service, please write directly to Accora at the address on the back cover. Provide the dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Accora will issue a return authorization. The defective unit or parts shall be cleaned using disinfectant prior to return and must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; DAMAGE CAUSED BY ANY EVENTS OUTSIDE THE CONTROL OF ACCORA INCLUDING, BUT NOT LIMITED TO: DAMAGE FROM CARELESSNESS, ACCIDENTAL DAMAGE, DAMAGE FROM SHARP OBJECTS, TEARS, NEGLIGENCE (WHETHER WILLFUL OR NOT), TAMPERING, DAMAGE FROM FIRE, DAMAGE OCCURRING IN TRANSIT AFTER DELIVERY BY ACCORA OR EXCESSIVE WEAR AND TEAR, INCLUDING BUT NOT LIMITED TO THE FAILURE TO FOLLOW CLEANING INSTRUCTIONS, USE OF INAPPROPRIATE CLEANING COMPOUNDS AND FAILURE TO ENSURE THAT ANY PRODUCT IS THOROUGHLY DRIED BEFORE TURNING OR FLIPPING IT OVER; DAMAGE RESULTING FROM CONTAMINATION FOLLOWING DAMAGE TO THE COVER AND FAILURE TO REPLACE IT, CONTAMINATION DUE TO INCORRECT STORAGE OR FAILURE TO INSPECT THE INTERIOR OF THE PRODUCT AS STATED HEREIN, OR DAMAGE DUE TO UNUSUAL CONTAMINATION OR INFECTION, SPILLAGE OR STAINING, OR USE OF ABRASIVE, CORROSIVE OR ANY SUCH SIMILAR DAMAGING SUBSTANCE; DAMAGE DUE TO UNDERTAKING ANY ACTION OR PROCEDURE USING THE PRODUCT OUTSIDE ITS NORMAL AND INTENDED USE; DAMAGE RESULTING FROM FAILURE TO REPLACE DAMAGED PARTS PROMPTLY OR DAMAGE DUE TO INTERIOR PARTS BEING USED WITHOUT THE COVER, BEING USED WITH A DAMAGED COVER, OR WITH THE COVER BEING INCORRECTLY FITTED; DAMAGE WHEN ANY PARTS OF THE PRODUCT ARE USED AS PART OF, OR IN CONJUNCTION WITH, ANY OTHER PRODUCT OR PIECE OF EQUIPMENT EXCEPT AS SPECIFICALLY PROVIDED FOR BY ACCORA; DAMAGE NOT PROVEN TO THE SATISFACTION OF ACCORA TO BE DUE TO DEFECTIVE MATERIALS, WORKMANSHIP OR ASSEMBLY; OR PRODUCTS MODIFIED WITHOUT ACCORA EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS OR PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF ACCORA. IT IS A CONDITION OF THIS WARRANTY THAT THE FOLLOWING IS COMPLIED WITH:

1) ALL INSTRUCTIONS FOR CLEANING AND MAINTENANCE OF THE PRODUCT ARE ADHERED TO.

2) THE INTERNAL AND EXTERNAL PARTS OF ANY PRODUCT ARE INSPECTED AT LEAST ONCE EVERY THREE (3) MONTHS FOR ANY DAMAGE AND WEAR. ALL DAMAGED PARTS ARE REPLACED IMMEDIATELY WITH THE APPROPRIATE REPLACEMENT PART, OR THE PRODUCT IS IMMEDIATELY TAKEN OUT OF SERVICE UNTIL THE REPLACEMENT PART HAS BEEN OBTAINED AND THE DAMAGED PART HAS BEEN REPLACED.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER

EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR, REPLACEMENT, OR REFUND OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. ACCORA SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER AND DISCLAIMS ANY LIABILITY BEYOND THE INITIAL PURCHASE PRICE OF THE PRODUCT. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/ PROVINCIAL LAWS AND REQUIREMENTS.

Technical Data

Materials

Top Cover	Polyfiber / Polyurethane Stretch
Bottom Cover	Vinyl

Dimensions

Part No	Length (in)	Width (in)
ACC400-0-FM1-100	80	36
ACC400-0-FM1-200	84	36
ACC400-0-FM2-100	80	42
ACC600-0-FM1-100	80	36
ACC600-0-FM1-200	84	36
ACC600-0-FM2-100	80	42

Contact Details

Manufactured By:

Proactive Medical, 270 Washington Street, Mount Vernon, NY 10553

Distributed By:

Accora Inc. 9210 Corporate Blvd. Suite 120, Rockville, MD 20850.

T: 301-560-2400 information@accora.us.com